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Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br
aditek@aditek.com.br
3. Intended Use:

The use of orthodontic Brackets Slide H5 Clear is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

The products have been developed for use in the oral cavity environment, directly on the teeth, with the objective of supporting orthodontic treatments in humans, promoting a better positioning of the teeth in the dental arches toward a correct position and occlusion. They are indicated to treat patients who have problems of malocclusion that require alignment and leveling movements of the teeth in the dental arch.

They are chosen by orthodontic professionals who seek a more aesthetic and faster clinical care in the exchange of orthodontic wires or archwires.

The brackets have a clip that holds the wire into the slot, eliminating the use of ligatures or ligature wires.

The brackets are used in conjunction with the archwires, wires, springs, bands, elastomerics and other auxiliary products for the orthodontic treatment.

The Slide H5 is designed to be used with light force, thermal memory archwire with a maximum gauge of up to .017 "x.025", as recommended in the table below.

Archwire Sequence recommended		
Phase I	Alignment Phase	Archwire deflection
<ul style="list-style-type: none"> Alignment and leveling Arch development 	.013" Contour NiTi .014" Contour NiTi or Thermo NiTi	Over 0,5mm
Phase II	Leveling Phase	
<ul style="list-style-type: none"> Complete rotation Torque control Angulation correction Level Spee Curve 	.016" Contour NiTi or Thermo NiTi .014"x.025" Contour NiTi .016"x.025" Contour NiTi .016"x.022" Nitinol Reverse Curve	0,35mm to 0,5mm
Phase III	Working Phase	
<ul style="list-style-type: none"> Space closing Anterior retraction with torque control 	.016"x.022" Thermo NiTi or .016"x.025" Contour NiTi	Less than 0,35mm

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Phase IV	Finishing Phase	
<ul style="list-style-type: none"> • Midline correction • Root alignment and final torque control 	.016"x.022" TMA .017"x.025" Thermo NiTi	Less than 0,15mm

NOTE: The choice of material to be used is made by the qualified professional, based on his knowledge acquired during his training in conjunction with the diagnosis and treatment plan for each patient.

4. Users:

The application of Brackets Slide H5 is restricted to dental professionals with qualification in Orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

5. Performance Characteristics:

Commonly orthodontic treatment without interruption takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral conditions of the patient, which can be extended according to the complexity of the case.

6. Clinical Benefits:

The main benefits that these systems bring to the life of the patient is the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

7. Warning:

Aditeks' products are designed and manufactured for single use only. The reuse or reprocessing may cause cross infection, decreased efficiency in bonding and loss of mechanical properties due to natural wear and forces applied in the removal of the product.

If the package has been breached, it is recommended to discard the product, minimizing the risk of transmission of microorganisms or possible product adulteration.

Store product in cool, dry and out of direct sunlight, keeping it in its own package until use.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy the woman becomes more susceptible to having periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure for bonding and removing the brackets from the teeth must be performed according to the instructions of the manufacturer of the acquired orthodontic adhesive.

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The composition of the product does not require specific procedures for transport and presents no risk to people.

The product is supplied in the non-sterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are necessary.

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Aditek do Brasil Ltda. is not responsible for the results obtained by using the product without necessary precautions or non-compliance with the warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil must be notified to the national competent authority in which the orthodontist is established.

8. **Contraindications:**

- Patients with poor oral hygiene,
- Patients with inability to cooperate with the treatment,
- Disease or limitation that might hinder treatment,
- Existing bone or root resorption,
- Existing decalcification of dental enamel,
- Existing periodontal complications,
- Complex treatments with extensive biomechanics.

It is the primary responsibility of the orthodontist to identify any possible contraindication that may preclude the use of the product.

9. **Residual Risks:**

All finished orthodontic treatments are subject to relapse. After treatment, the control must be determined by the professional to avoid or minimize relapse

10. **Side Effects:**

The use of the medical device may result in discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, oral and mucosal damage, difficulty in speaking or chewing, discomfort and pain.

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7 years

13. Symbols and Meanings:

	Manufacturer		Product Reference Code
	Consult instructions for use		Expiration date
	Attention		Do not reuse product
	Do not use if the package is damaged or breached		Lot number

14. Patient information:

Perform daily good hygiene to prevent plaque installation and subsequent periodontal disease and to remove food particles that may settle under the clip. The use of an oral irrigator, type Water Pik, is recommended to help cleaning the teeth and brackets;

It is important that the pregnant patient be referred for professional cleaning at regular intervals, due to the increased susceptibility of periodontal problems during pregnancy;

It is important to be careful when chewing certain types of food, especially the hard or crunchy ones as they can take off or break the brackets. These foods should be cut before eating. Hard foods such as popcorn and ice should be avoided.

Do not place objects in the mouth, such as pens and pencils, and be careful not to bang or bite the brush when brushing your teeth, which may break or loosen the bracket when subjected to a lever motion;

Foods and / or drinks with a lot of coloring, natural or not, favor the staining of the brackets; When taking drinks such as wine, coffee, acai, soft drinks, you should rinse your mouth with water after ingesting them;

Smoking or chewing tobacco can stain your brackets;

In case of damage, look for an orthodontist;

If swallowed or aspirated, look for medical care;

When participating in sports, always wear mouth guards, as impacts can damage appliances and harm in the oral cavity;

Mention all the undesirable side effects;

Mention about the risk of recurrence after treatment;

The patient must follow the professional's instructions regarding retainer to avoid recurrence.

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
The steps below must be performed only by dental professionals with qualification in Orthodontics.


Prepare the tooth surface according to the instructions of the manufacturer of the acquired orthodontic adhesive.

Bond the brackets on the teeth according to the instructions of the manufacturer of the acquired orthodontic adhesive.

We suggest the use of adhesives UltraBond (Aditek) e LightBond (Aditek).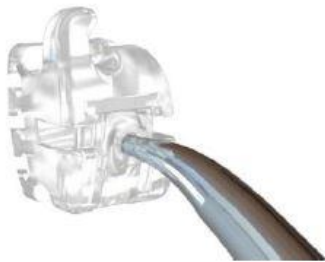


Insert the instrument in the appropriate place in the center of the clip;

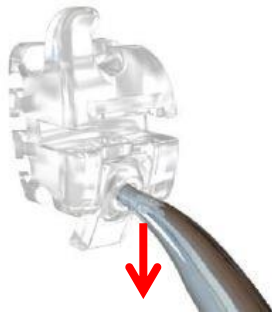Gentle pull the instrument vertically to occlusal.
Excessive force may cause the clip to fail;

Insert the selected archwire according to the stage of the treatment;

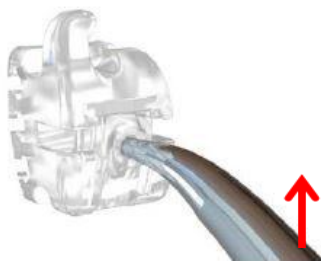To close, position the instrument and move the clip in the cervical direction
It is important to feel the clip locking over the protuberance and hear the audible click.

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Note: Be careful when attaching the elastomeric so as not to place the instrument under the wing of the bracket exerting a lever motion, which may cause the wing breakage.

Removing the bracket

Remove the resin around the bracket with a multilaminated burr for adhesive removal. Place the instrument to remove the bracket in the distal mesial direction and make a clockwise movement. The bracket should be loose, leaving a small amount of adhesive on the tooth, which must be removed with a high speed multilaminated burr for resin or plier to remove resin.

NOTE: If it is difficult to remove the bracket, cut the bracket horizontally at the height of the slot with a diamond burr, taking care not to touch the tooth enamel. After sectioning, remove the parts with an appropriate instrument.

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